

# Dental Autoclave



## ►► Features:

- Fully Automatic
- 8 Preset sterilization programs
- Ster. Temp.: 105-134
- Ster. Time: 4-60 min
- Dry time: 1-25min
- Vacuum Times: 1-10 times
- B&D / Helix / Vacuum testing program
- Cleaning Program
- Printer & USB Output
- Big Stainless Steel Water Tanks
- High quality bacteria filter

Program	121°C Solid	121°C Hollow	121°C Porous	121°C Solid	121°C Hollow	121°C Porous
Avacuum Times	1	3	3	1	3	3
Temp (°C)	121	121	121	134	134	134
Pressure(KPA)	110	110	110	210	210	210
Time(min.)	20	20	20	4	4	4
Program	Cotton	Prion	User defined	BD/Helix Test	Vacuum Test	Clean
Vacuum Time	3	3	User defined	3	1	3
Temp (°C)	134	134	User defined	134	/	105
Pressure(KPA)	210	210	User defined	210	-80	20
Time(Min.)	4	19	User defined	3.5	15	5



## ▶▶▶ Technical Specification

Description	Small Autoclave
Class	B
Certification	CE 1023, ISO 13485
Standard	BS EN 13060:2004 + A2:2010
Frequency	50Hz
Plug	3 hole plug
Wattage	2.4 KW
Sterilization temperature	105 - 134°C
Max working pressure	0.21-0.23MPA
Life span	5 Years
Diameter of chamber	20 CM
Depth of chamber	36 CM
Volume of chamber	22L
Material of chamber	Stainless steel 304
Colors available	White
Screen type	LCD
Screen size	6 X 4.25 cm
Language available	English
Lock system of door	Mechanic + electrical lockers
User password	Yes
Output	Thermo printer & USB
Net weight	51 kgs
Gross weight / carton	59 kgs
Product size	61 X 50 X 43cm
Package size	73X 57 X 51cm
Min size of table for installation	54 x 41.5 cm

## Support & Services

### ▶▶▶ HPLC Servicing, Validation, Trainings and Preventive Maintenance :

- HPLC Servicing** :We have team of service engineers who can attend to any make of HPLC promptly @the most affordable cost.
- Trainings** :We also take up preventive Maintenance to reduce downtime of HPLC's
- AMC's/CMC** :We offer user training both in-House and at customer sites on HPLC principles, operations, troubleshooting.
- Validations** :We have protocols for carrying out periodic Validations as per GLP/GMP/U SFDA norms
- Instruments** :We offer instruments / Renting Services Modules like pumps, detector etc. on Rent.



## ▶▶ About Analytical Technologies

Analytical Technologies is synonymous for offering technologies for doing analysis and is the Fastest Growing Global Brand having presence in at least 96 countries across the globe. Analytical Technologies Limited is an ISO :9001 Certified Company engaged in Designing, Manufacturing, Marketing & providing Services for the Analytical, Chromatography, Spectroscopy, Bio Technology, Bio Medical, Clinical Diagnostics, Material Science & General Laboratory Instrumentation. Analytical Technologies, India has across the Country operations with at least 4 Regional Offices, 6 Branch Offices & Service Centers. Distributors & Channel partners worldwide.

## ▶▶ About Analytical Technologies



UV/VIS Spectro 2060+ Double Beam



PTIR



Gas Chromatograph 3000



Gas Chromatograph 2979 Plus



Flash Chromatograph



Atomic Absorption Spectrophotometer



Liquid Partical Counter



Optical Emission Spectrophotometer



DSC/TGA



NOVA 2020 plus Automated Bio Chemistry Analyzer



HEMA 2020 Hematology Analyzer



Micro Plate Reader/Washer



Water purification system



Total Organic Carbon



Fully Automated CLIA



NOVA Basic Semi -Auto Chemistry Analyzer



PCR/Gradient PCR/ RTPCR



Blood Gas Analyzer



Random access Analyzer for immunoassay Proteins & clinical chemistry



Semen Analyzer



Water Purification System

▶▶▶ Regulatory compliances



▶▶▶ Corporate Social Responsibility

Analytical Foundation is a Nonprofit Organization (NGO) found for the purpose of:



**Analytical**  
**Foundation**

1. Research & Innovation Scientist's awards / QC Professional Award : Quality life is possible by innovation only and the innovation is possible by research only, hence ANALYTICAL FOUNDATION is committed to identify such personalities for their contributions across various field of Science and Technology and awarding them yearly. To participate for award, send us your details of research / testing / publication at [info@analyticalfoundation.org](mailto:info@analyticalfoundation.org)

2. Improving quality of life by offering YOGA Training courses, Work shops / Seminars etc.

3. ANALYTICAL FOUNDATION aims to DETOXYFY human minds, souls and body by means of Yoga, Meditation, Ayurveda, Health Care, Awards, Media, Events, Camps etc.

▶▶▶ Reach us @



HPLC Solutions   MultipleLabs   Analytical Bio-Med   Analytical Distributors   Analytical Foundation (Trust)

**Corporate & Regd. Office:**  
Analytical House, # E67 & E68,  
Ravi Park, Vasna Road, Baroda,  
Gujarat 390 015. INDIA

T: +91 265 2253620  
+91 265 2252839  
+91 265 2252370  
F: +91 265 2254395

E: [info@hplctechnologies.com](mailto:info@hplctechnologies.com)  
[info@multiplelabs.com](mailto:info@multiplelabs.com)  
[info@analyticalgroup.net](mailto:info@analyticalgroup.net)  
[info@analyticalbiomed.com](mailto:info@analyticalbiomed.com)

W. [www.ais-india.com](http://www.ais-india.com)  
[www.analyticalgroup.net](http://www.analyticalgroup.net)  
[www.hplctechnologies.com](http://www.hplctechnologies.com)  
[www.multiplelabs.com](http://www.multiplelabs.com)

Sales & Support Offices:  
across the country :  
Distributors & Channel  
partners World Wide

Note : Company reserves rights to add/delete/modify the contents / technical specifications of the catalogue without prior notice.