

# STERNAL SAW

## SS2000




EPC / PRODUCTS / APPLICATION / SOFTWARE / ACCESSORIES / CONSUMABLES / SERVICES

**Analytical Technologies Limited**

An ISO 9001 Certified Company

[www.analyticalgroup.net](http://www.analyticalgroup.net)

## ►► USING THE CHARGER

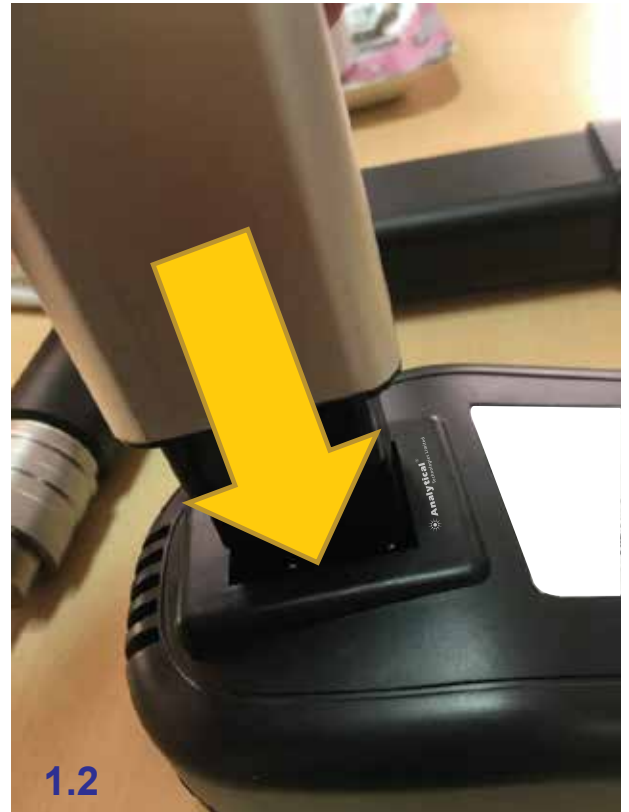

- STEP 1** : PLUG THE CHARGER IN A SOCKET.CHECK THE POWER ON LED LIGHT GOES ON.
- STEP 2** : SEE THE POINTS COMING OUT OF THE CHARGER SHOWN IN PICTURES 1.1
- STEP 3** : ATTACH THE BATTERY FROM THE SAME SIDE GENTLY SHOWN IN PICTURES 1.2
- STEP 4** : PUSH IT INSIDE AT ONCE SHOWN IN PICTURES 1.3
- STEP 5** : WAIT TILL THE LED LIGHT FOR CHARGING GOES GREEN.

## ►► INSERT/REMOVE BLADE



- STEP 1** : REMOVE THE BATTERY AS SHOWN IN 4.1
- STEP 2** : REMOVE THE STERNAL GUARD BY USING EASY MOVING THE METAL BRACELET BACKWARD AS SHOWN IN PICTURE 2.2
- STEP 3** : TWIST THE OUTER RING TILL THE LINE GETS STRAIGHT WITH THE BLADE AS SHOWN IN PICTURE 2.5
- STEP 4** : PULL OUT THE BLADE
- STEP 5** : DO THE SAME FOR INSERTING A BLADE

## ►► INSERT BATTERY USING ASEPTIC TRANSFER KIT



- STEP 1** : AUTOCLAVE THE ASEPTIC TRANSFER KIT
- STEP 2** : SLIDE THE ASEPTIC TRANSFER KIT TO THE BATTERY WITH HELP OF THE GROVES SHOWN IN PICTURES 3.1 AND 3.2
- STEP 3** : THEN HOLD IT THROUGH THE RING ABOVE THE BATTERY IT DOES NOT SLIDE DOWN SHOWN IN PICTURES 3.3.
- STEP 4** : WATCH FOR THE SAME GROVES IN BATTERY AND IN STERNAL SAW WHERE BATTERY IS INSERTED SHOWN IN PICTURES 3.4.
- STEP 5** : RELEASE THE RING AND THE BATTERY SLIDES DOWN THE GROVES SHOWN IN PICTURES 3.5
- STEP 6** : ATTACH THE COVER SHOWN IN 4.1 AND 4.2.

## ►► REMOVE BATTERY



- STEP 1** : SWITCH THE ARROW FROM CLOSE TO OPEN AS SHOWN IN PICTURES 4.1,4.2 AND 4.3.
- STEP 2** : OPEN THE RING ATTACHED ABOVE THE BATTERY GENTLY AS SHOWN IN PICTURES 4.4 AND 4.5.
- STEP 3** : WITH THE HELP OF IT OPEN PULL OUT THE BATTERY PICTURE 4.6.

## ►► USING THE HAND PIECE



PLEASE CHECK WHETHER THE BLADE IS ATTACHED AND THEN BATTERY IS FULLY CHARGED.

PLACE THE MIDDLE FINGER BELOW THE SPEED CONTROL AS GRIP OF VARIABLE SPEED BUTTON GETS BETTER. THE GENTLY INCREASING THE PRESSURE ON THE BUTTON REACH TO SPEED AT WHICH THE DOCTOR WANTS TO OPERATE.

THEN PLACE IT NEAR THE STERNAL AND SLIDE IT AT ONCE WITH FORCE.



## ►► HPLC Servicing, Validation, Trainings and Preventive Maintenance :

HPLC Servicing	:HPLC Servicing : We have team of service engineers who can attend to any make of HPLC promptly @the most affordable cost.
Trainings	:We also take up preventive Maintenance to reduce downtime of HPLC's Trainings.
AMC's/CMC	:AMC's/CMC :We offer user training both in-House and at customer sites on PLC principles, operations, troubleshooting.
Validations	:Validations :We have protocols for carrying out periodic Validations as per GLP/ GMP/U SFDA norms.
Instruments	:Instruments :We offer instruments / Renting Services Modules like pumps, detector etc. on Rent.



## ►► About Analytical Technologies

Analytical Technologies is synonymous for offering technologies for doing analysis and is the Fastest Growing Global Brand having presence in at least 96 countries across the globe. Analytical Technologies Limited is an ISO :9001 Certified Company engaged in Designing, Manufacturing, Marketing & providing Services for the Analytical, Chromatography, Spectroscopy, Bio Technology, Bio Medical, Clinical Diagnostics, Material Science & General Laboratory Instrumentation. Analytical Technologies, India has across the Country operations with at least 4 Regional Offices, 6 Branch Offices & Service Centers. Distributors & Channel partners worldwide.

## ►► Our Products & Technologies



## ►► Regulatory compliances



## ►► Corporate Social Responsibility

Analytical Foundation is a Nonprofit Organization (NGO) found for the purpose of:



**Analytical**  
**Foundation**

1. Research & Innovation Scientist's awards / QC Professional Award : Quality life is possible by innovation only and the innovation is possible by research only, hence ANALYTICAL FOUNDATION is committed to identify such personalities for their contributions across various field of Science and Technology and awarding them yearly. To participate for award, send us your details of research / testing / publication at [info@analyticalfoundation.org](mailto:info@analyticalfoundation.org)

2. Improving quality of life by offering YOGA Training courses, Work shops / Seminars etc.

3. ANALYTICAL FOUNDATION aims to DETOXYFY human minds, souls and body by means of Yoga, Meditation, Ayurveda, Health Care, Awards, Media, Events, Camps etc.

## ►► Reach us @



 **Analytical**®  
Technologies Limited

HPLC Solutions   MultipleLabs   Analytical Bio-Med   Analytical Distributors   Analytical Foundation (Trust)

**Corporate & Regd. Office:**  
Analytical House, # E67 & E68,  
Ravi Park, Vasna Road, Baroda,  
Gujarat 390 015. INDIA

T: +91 265 2253620  
+91 265 2252839  
+91 265 2252370  
F: +91 265 2254395

E: [info@hplctechnologies.com](mailto:info@hplctechnologies.com)  
[info@multiplelabs.com](mailto:info@multiplelabs.com)  
[info@analyticalgroup.net](mailto:info@analyticalgroup.net)  
[info@analyticalbiomed.com](mailto:info@analyticalbiomed.com)

W. [www.ais-india.com](http://www.ais-india.com)  
[www.analyticalgroup.net](http://www.analyticalgroup.net)  
[www.hplctechnologies.com](http://www.hplctechnologies.com)  
[www.multiplelabs.com](http://www.multiplelabs.com)

Sales & Support Offices:  
across the country :  
Distributors & Channel  
partners World Wide

Note : Company reserves rights to add/delete/modify the contents / technical specifications of the catalogue without prior notice.