

VSS - 3267

Seal Safe Automatic Fixative Filling And
Vacuum sealing system for Biospecimen



EPCC / PRODUCTS / APPLICATION / SOFTWARE / ACCESSORIES / CONSUMABLES / SERVICES

Analytical Technologies Limited

An ISO 9001 Certified Company

►► SealSAFE

Automation, standardization and safety for full biospecimen management

The SealSafe is the world's first automatic system that fills dedicated bags containing the biospecimens, with the required fixative (i.e. formalin), in a standardized and documented way. The innovative (patent applied for)* SealSAFE technology, allows operators at surgery rooms and at pathology laboratories, to handle histology specimens in full safety. Today's manual operation to prepare, transport and stock histology specimens, can be drastically improved with this disruptive technology.



Place specimen in the vacuum bag and introduce it into the SealSAFE cavity. Close the lid.

A load cell in the cavity automatically weighs the specimen. According to a preset ratio (1:1, 1:2, 1:2.5, 1:3) fixative is then injected into the bag.



As a final step the SealSAFE vacuum seals the specimen bag. Fixative injection and sealing process are completed in the closed and ventilated cavity.


The sealed bag containing the specimen with full documentation is transported to the pathology lab in full safety. The same bag can be resealed up to three times from the operating theatre to final storage.




►► INNOVATIVE WORKFLOW

The traditional procedures of collection, transportation and archiving of pathology specimens are specific at each institution.


The introduction of SealSAFE technology at the starting point of sample generation (the surgery room), pathology accessioning, and in the grossing area provides a dynamic and flexible solution with benefits to all involved departments.

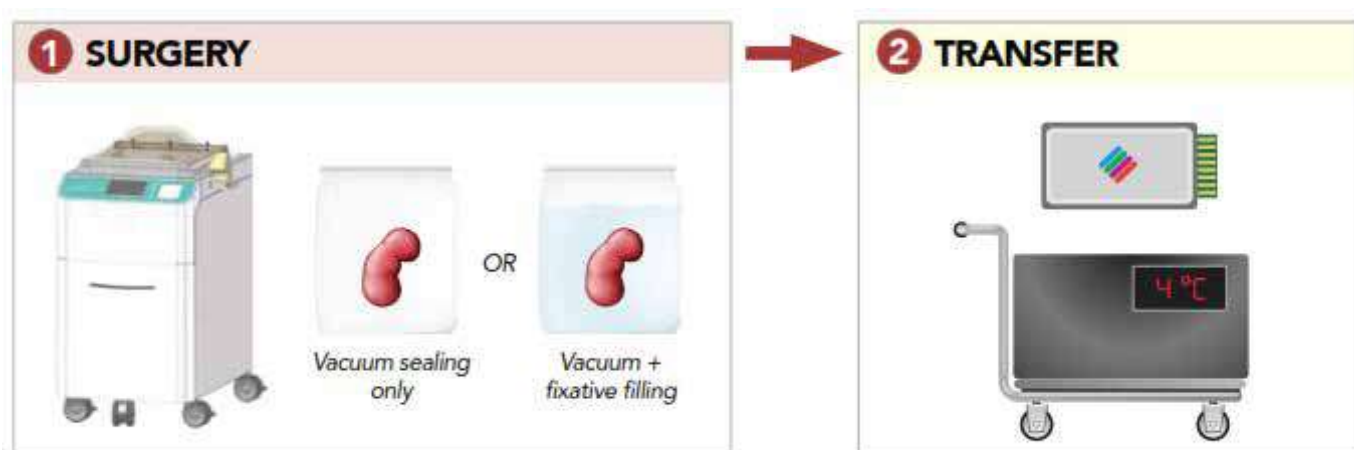


In the dirty room of a surgery department, the SealSAFE is used to vacuum seal biospecimens for transportation to the pathology laboratory. The operator can select the desired procedure for specimen management: fixative-free vacuum sealing or automated filling of fixative plus vacuum sealing. Time and temperature conditions during transportation are monitored with a dedicated electronic data-logger, activated by operator at the SealSAFE starting point. In case of vacuum sealing without fixative, transportation and storage at +4°C are required. In this modality, optimal preservation of the molecular material of the specimen is guaranteed up to 72 hours.

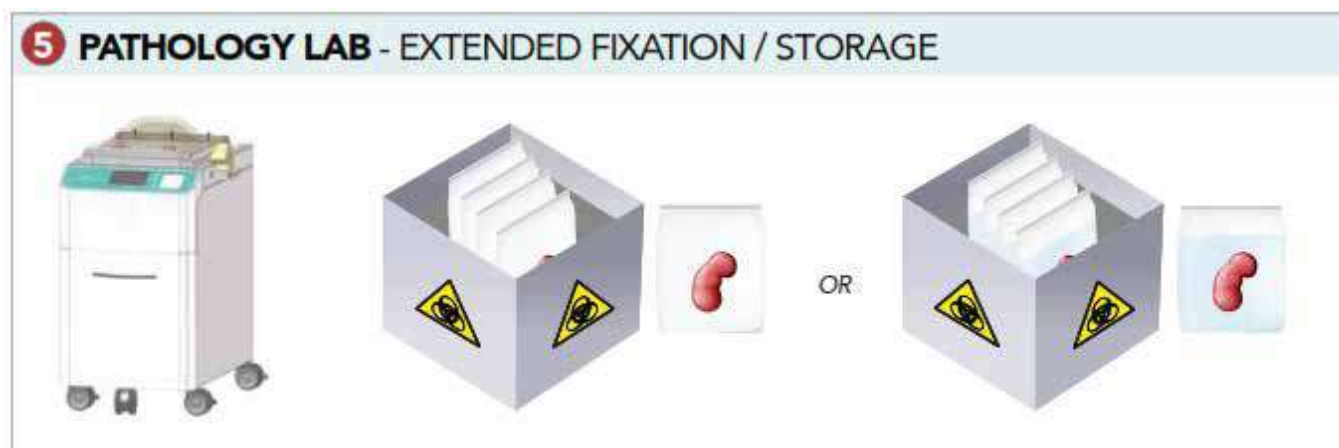

Installing a SealSAFE unit in the pathology laboratory enables maximum flexibility for specimen management. In addition, SealSAFE reusable vacuum storage bags allow for multiple opening and sealing events, which can accommodate: A. status pre-examination of large case specimen fixation/hardening; B. receipt, grossing, and final storage of medium-sized specimens; C. well fixed tissues can be sealed with minimal amounts of formalin or simply with vacuum to achieve drastic cost and space savings.

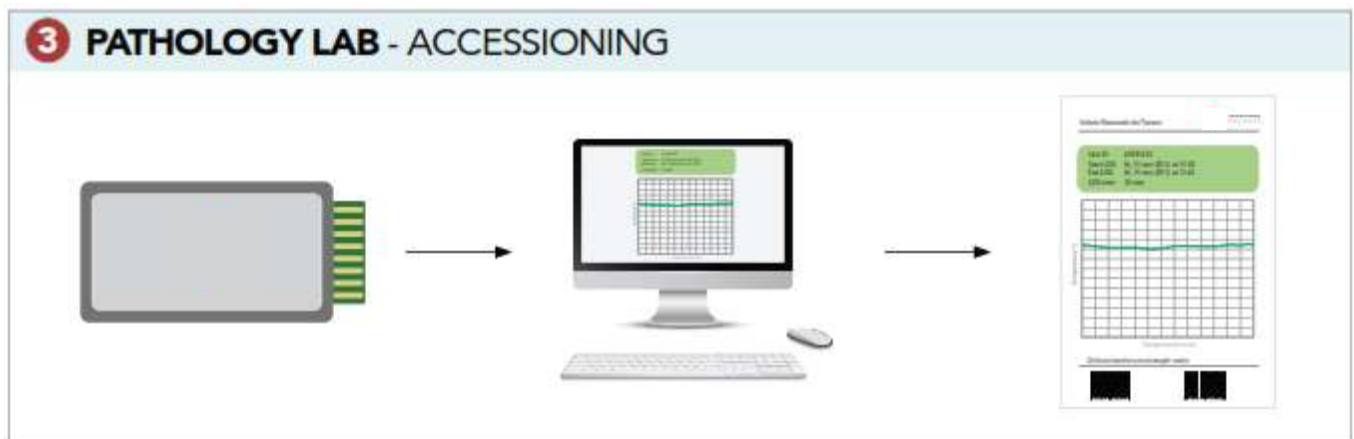

As specimens arrive into pathology and are processed through accessioning, relevant data can be downloaded and saved for future use. A detailed profile of transportation temperature conditions, duration of storage, and additional identification information is viewable and easily associated with patient cases.

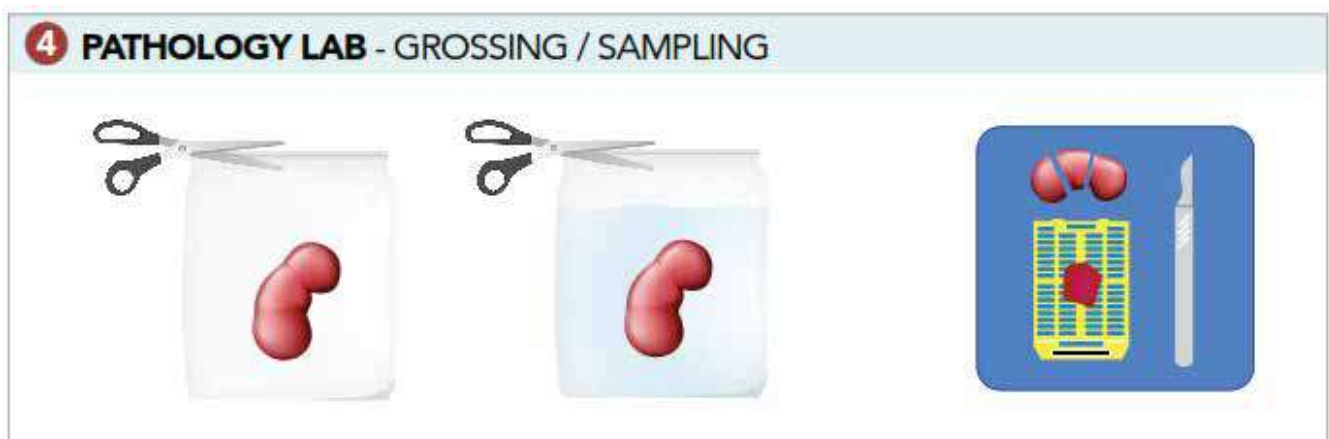

At the pathology laboratory, the specimen is received in a standardized fixative quantity or in “as fresh” conditions. The sampling and grossing procedure are therefore carried out without necessary modification of the current procedures.

►► BENEFITS

- **Standardization and Quality**

Always the right amount of formalin according to the specimen weight.

Optimized fixation and histological results.

- **Less formalin**

Reduce the volume of this toxic reagent through more efficient and standardized formalin/specimen ratios.

- **Safety**

Full automation and closed operation prevents exposure to formalin fumes.

- **Improved archiving**

Storage in vacuum bags dramatically reduces volume.

Archiving even without formalin is possible.

- **Extreme cost saving for disposal**

Lower disposal cost thanks to a decreased volume of bio-hazard material with vacuum bags.

►► SPECIAL BAGS

- Double layered PA/PE, 135 microns in thickness guarantees greater mechanical flexibility and resistance to perforation. Available in five different dimensions, suitable for all kind of specimens.

- Three sealing areas and cutting lines marked for multiple reuse.

- Sealable document pouch provides security for patient information.

- Automatically printed adhesive label contains: ID, date, time, specimen and fixative weight.

- Enlarged base for self-standing.





HPLC Servicing, Validation, Trainings and Preventive Maintenance :

HPLC Servicing :HPLC Servicing : We have team of service engineers who can attend to any make of HPLC promptly @the most affordable cost.

Trainings :We also take up preventive Maintenance to reduce downtime of HPLC's Trainings.

AMC's/CMC :AMC's/CMC :We offer user training both in-House and at customer sites on HPLC principles, operations, trouble-shooting.

Validations :Validations :We have protocols for carrying out periodic Validations as per GLP/GMP/USFDA norms.

Instruments :Instruments :We offer instruments/Renting Services Modules like pumps,detector etc. on Rent.



About Analytical Technologies

Analytical Technologies is synonymous for offering technologies for doing analysis and is the Fastest Growing Global Brand having presence in at least 96 countries across the global. Analytical Technologies Limited is an ISO:9001 Certified Company engaged in Designing, Manufacturing, Marketing & providing Services for the Analytical, Chromatography, Spectroscopy, Bio Technology, Bio Medical, Clinical Diagnostics, Material Science & General Laboratory Instrumentation. Analytical Technologies, India has across the Country operations with at least 4 Regional Offices, 6 Branch Offices & Service Centers. Distributors & Channel partners worldwide.

Our Products & Technologies



UV/VIS
Spectro 2080+
Double Beam



Infra FTIR



Optima Gas
Chromatograph
3007



Optima Gas
Chromatograph
2979 Plus



Flash
Chromatograph



Atomic Absorption
Spectrophotometer



Liquid Particle
Counter



Optical Emission
Spectrophotometer



DSC/TGA



Semi Auto Bio
Chemistry Analyzer



HEMA 2062
Hematology
Analyzer



Micro Plate
Reader/Washer



URINOVA 2800
Urine Analyzer



Total Organic
Carbon 3800



Fully Automated
CLIA



NOVA-2100
Chemistry Analyzer



PCR/Gradient PCR/
RTPCR



TOC
Analyzer



Laser Particle
Size Analyzer



Ion Chromatograph



Water purification
system

Regulatory compliances



Corporate Social Responsibility

Analytical Foundation is a nonprofit organization (NGO) found for the purpose of:



Analytical Foundation

1. Research & Innovation Scientist's awards/QC Professional Award : Quality life is possible by innovation only and the innovation is possible by research only, hence ANALYTICAL FOUNDATION is committed to identify such personalities for their contributions across various field of Science and Technology and awarding them yearly. To participate for award, send us your details of research / testing / publication at Info@analyticalfoundation.org

2. Improving quality of life by offering YOGA Training courses, Work shops/Seminars etc.

3. ANALYTICAL FOUNDATION aims to DETOXYFY human minds,souls and body by means of yoga, Meditation, Ayurveda, Health Care, Awards, Media, Events, Camps etc.



HPLC Solutions MultipleLabs Analytical Bio-Med Analytical Distributors Analytical Foundation (Trust)

Corporate & Regd. Office:
Analytical House, # E67 & E68,
Ravi Park, Vasna Road, Baroda,
Gujarat 390 015. INDIA

T: +91 265 2253620
+91 265 2252839
+91 265 2252370
F: +91 265 2254395

E: info@hplctechnologies.com
info@multiplelabs.com
info@analyticalgroup.net
info@analyticalbiomed.com

W. www.analyticalgroup.net
www.hplctechnologies.com
www.multiplelabs.com
www.ais-india.com

Sales & Support Offices:
across the country :
Distributors & Channel
partners World Wide